

**Warranty: 14 days from date put into use**

## Read instructions thoroughly before use.

This pad must be discarded and replaced after warranty expires or at any time that it is no longer functional. Failure to do so could result in false or no alarms, leading to injury or death of a person in your care.



## WARNINGS

- Store flat. Do not fold or crease pad.
- Do not tamper with sensor.
- Do not connect sensor to telephone jack.
- Never connect to another manufacturer's control unit.
- Do not submerge or expose pad to excessive moisture or fluids.
- Do not sterilize with heat, gas, or radiation. Clean by wiping with a non-caustic solution.
- Sensor pad is for single patient use only. Do not use for multiple patients at any time.
- May not be suitable for all high-risk patients. Sensors can be defeated by those who are cognitively aware, subject to moments of lucidity, or uncooperative. Review your facility's criteria to determine appropriate patient or resident applications.
- Device is not a substitute for routine visual monitoring.



## SETTING UP THE COMMODE PAD

- 1) Secure the pad to the underside of the commode or toilet seat by removing paper backing located on the bottom of the pad and pressing firmly to ensure proper adhesion. Be sure the pad is located directly above a commode support or the toilet bowl rim.
- 2) Test system performance before first use and before each use thereafter.

## TEST THE PAD BEFORE EACH USE

- 1) Connect the pad to a Bed-Check brand monitor (control unit).
- 2) Press firmly on the pad for 3 seconds by applying pressure to the top of the commode or toilet seat. When you remove pressure, the audio alarm should sound.
- 3) Reset control unit to silence audio alarm.
- 4) If Nurse Call is used, make sure connections are properly made and are functioning.
- 5) Refer to the User Manual of the control unit for additional information.