




**A FOR BED**  
Remove the Bed-Check® Sensormat® pad from plastic bag. Make sure all sheets and covers are pulled down. Place the Sensormat pad 3 to 5 inches below the bend in the mattress. Run the cord under the mattress toward the Classic-Check control unit, then plug cord into it.  
**OPTION:** Place Sensormat pad under the individual's shoulders.



**B FOR CHAIR**  
Place on the seat of the chair 4 inches from the back of the chair, across the width of the seat.

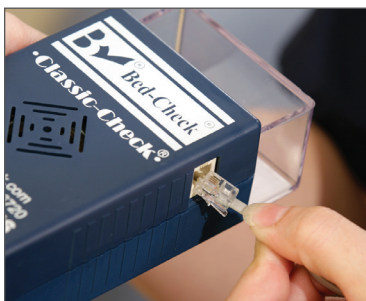

**C** Plug Sensormat pad into the Classic-Check control unit.  
**IMPORTANT:** Use only Sensormat pads manufactured by Bed-Check.




**D** Plug the Classic-Check control unit's power supply cord into the electrical outlet.




**E** Plug the nurse call cord into the facility nurse call interface. **NOTE:** An adaptor may be required. Consult your distributor.




**F** To secure the Classic-Check control unit out of the reach of the individual, attach the control unit to the bed frame\* using the standard Velcro® strap, or to the headboard using an optional wire holster.  
*\*To avoid urine damage, do not place directly under the mattress fold.*

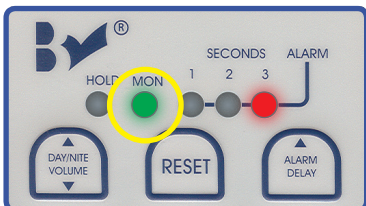


**G** Place individual in bed. The yellow HOLD light will flash for 5 seconds, then beep once as green MON light illuminates – the individual is being monitored. To immediately monitor, press RESET once. **Verify MON light is on before leaving individual unattended.**

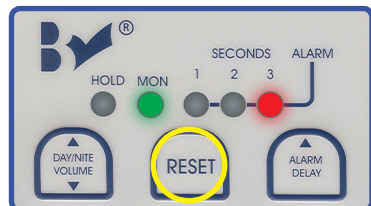

**H** When the individual attempts to exit the bed, an alarm will sound in the room and at the nurse station. Attend to the individual, then press RESET once to cancel the alarm. Be sure to reset the nurse call station in the individual's room.

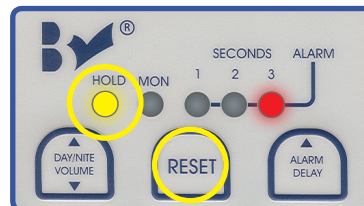

**I** To remove the individual from the bed or chair **without the alarm sounding**, press RESET. The yellow HOLD light will flash. Now assist individual from the bed or chair. The control unit remains in HOLD until weight returns to the Sensormat pad.

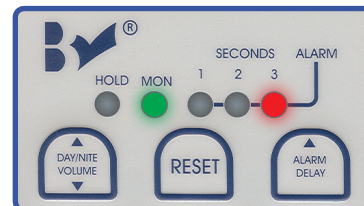

**J** If the individual has not been removed from the bed or chair within 25 seconds **after pressing RESET**, the control unit will return to the green MON (monitor) mode **automatically**. **NOTE:** Instantly go from MON to HOLD or from HOLD to MON by pressing RESET.

## IMPORTANT

- Check for system effectiveness daily and/or at shift change.
- Always check for correct positioning of Sensormat pad after any alarm.
- Use Bed-Check products only to maintain warranty and ensure compatibility.
- The physical characteristics, habits and factors that make each person at risk for falls are different and may change daily. Please reassess as necessary.



## Placement of the Sensormat® Pad

### IN THE BED

The Sensormat pad can be placed under the hips, with a 1-3 second delay setting recommended.


Setting a 2 or 3 second delay allows in-bed movement.

The Sensormat pad also can be placed under the shoulders, with a 3 second delay recommended.

### IN THE CHAIR

**Setting a 1 second delay is recommended.** Place the Sensormat pad widthwise toward the back of the chair, or if the individual is slouching, toward the front of the chair.

Positioning the Sensormat pad lengthwise on the chair surface allows the individual to move backward and forward without the alarm sounding.

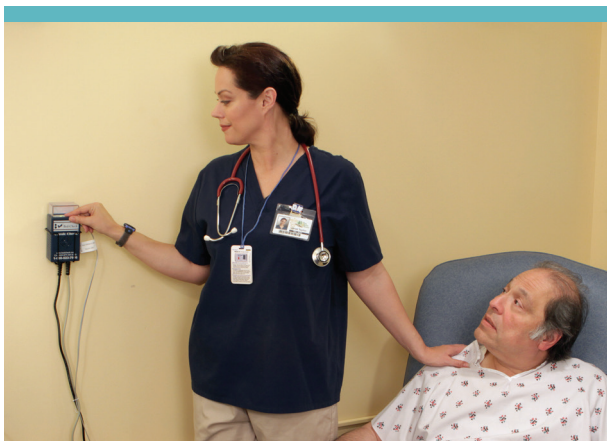

## Removing the Individual While the Control Unit is in Monitoring Mode

To remove an individual from bed without setting the alarm off, assist the individual to the sitting position, ready to stand (if possible).

Push RESET on the face of the unit. The yellow **HOLD** light will blink as the individual's weight is still on the Sensormat pad. Once RESET is pushed, you have 25 seconds to stand the individual. If more time is needed, push RESET again. If you do not get the individual up within the allotted time, the control unit will reset to the green **MON** mode. This is a safety feature to ensure individuals are monitored. **Sometimes nurses intend to remove an individual, but are called out of the room on an emergency. If this occurs the control unit, still sensing weight, will automatically reset to monitoring.**

**NOTE:** If the alarm sounds because the individual stood up after 25 seconds, push RESET to cancel. The control unit will remain in an indefinite HOLD until weight is reapplied to Sensormat pad.



## PT/OT Working with the Individual

To work with an individual, assist them to the sitting position, ready to stand (if possible).

Push RESET. You will notice that HOLD begins to blink as the individual's weight is still on the Sensormat pad. Once RESET is pushed, you have 25 seconds to stand the individual. If more time is needed, push RESET again.

**OPTION 1:** Move the individual left or right so that they are no longer on the Sensormat pad. In doing this, the individual is no longer having contact with the Sensormat pad.

**OPTION 2:** Move the individual slightly to the left or right, pull the Sensormat pad out from under the individual, and move it away from them.

To resume monitoring, lay individual back down and pull the Sensormat pad back into position.

**Always verify the green **MON** light is on before leaving an individual unattended.**