




Bed-Check® Classic-Check® Quick Programming Guide

This guide will assist you with manual programming of the Bed-Check® Classic-Check® control unit. More detailed manual programming instructions can be found in the User's Guide booklet.

For programming, only the control unit and power cord are needed. (Neither a Sensormat® nor the nurse call cord need to be plugged in during programming.)

Arming/Disarming System

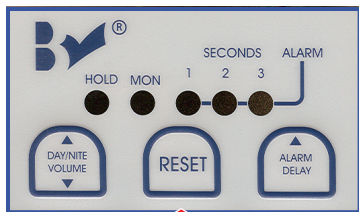


Press and release

Press and release the RESET button to go from MONITOR to HOLD, or from HOLD to MONITOR, or to silence an alarm. Please note that if the system is on HOLD, it will reset itself and resume monitoring if weight is placed on the Sensormat®. In addition, if the RESET button is pressed to put on HOLD and weight remains on the Sensormat®, the unit will return to monitoring mode in 25 seconds.


SETTINGS - Any of the following setting changes may be made independently of the others. We recommend setting tone before volume, as some tones may seem louder than others.

Day/Night Mode Settings

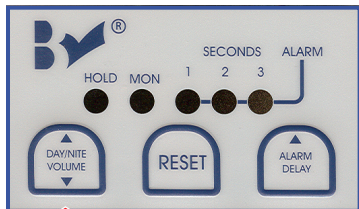

Press and release

Press and release DAY/NITE button to switch between DAY and NITE modes. NITE mode is indicated by a blue light in the splash guard.

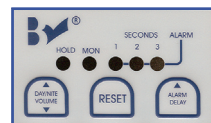


DAY Mode



NITE Mode

Alarm Delay Settings

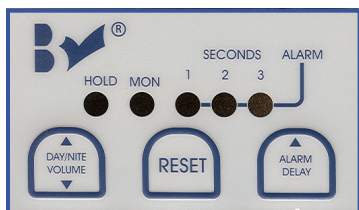

Press and release

Press and release the ALARM DELAY button to adjust the alarm delay to 1, 2, or 3 seconds.



1 second



2 seconds




3 seconds

Classic-Check®

Quick Programming Guide continued

Tone Setting

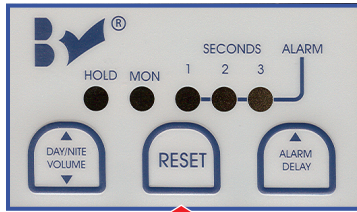


- ↑ Press and hold
- ↑ Press and hold
- ↑ Release; then Press/Release
- Release all
- ↑ Press and release to lock

The tone setting will be the same for DAY and NITE modes. To change the tone press *and hold* the RESET button, then press and hold the ALARM DELAY button. Release the RESET button to hear current setting. Then press and release the RESET button until desired tone is obtained. Release all buttons, press and release RESET once more to lock setting.

Display	Tone
● ● ● ● ● ● ● ● ● ●	Low Pitch Warble
● ● ● ● ● ● ● ● ● ●	High Pitch Warble
● ● ● ● ● ● ● ● ● ●	Triple Tone
● ● ● ● ● ● ● ● ● ●	Old McDonald
● ● ● ● ● ● ● ● ● ●	Mary Had a Little Lamb
● ● ● ● ● ● ● ● ● ●	The Farmer in the Dell
● ● ● ● ● ● ● ● ● ●	Comin' Round the Mountain
● ● ● ● ● ● ● ● ● ●	Tom, Tom the Piper's Son
● ● ● ● ● ● ● ● ● ●	Folk Song
● ● ● ● ● ● ● ● ● ●	Do - Re - Mi

Alarm Volume

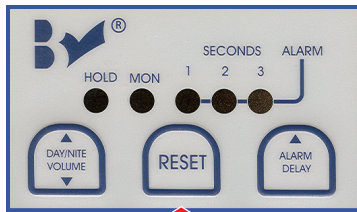

- ↑ Press and hold
- ↑ Press and hold
- ↑ Release; then Press/Release
- Release all
- ↑ Press and release to lock

The alarm volume can be adjusted separately for both the DAY and NITE modes. First set the mode for which you are changing the volume (see other side). Press *and hold* the RESET button, then press and hold the DAY/NITE VOLUME button. Release the RESET button to hear current setting. Then press and release the RESET button until desired volume is obtained. Release all. Press/release RESET once more to lock the setting.

Display	Decibels
● ● ● ● ● ● ● ● ● ●	67 db
● ● ● ● ● ● ● ● ● ●	73 db
● ● ● ● ● ● ● ● ● ●	77 db
● ● ● ● ● ● ● ● ● ●	80 db
● ● ● ● ● ● ● ● ● ●	81 db
● ● ● ● ● ● ● ● ● ●	83 db
● ● ● ● ● ● ● ● ● ●	85 db